

varioMAXI

Bedienungsanleitung
Operating instructions
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Gebruiksaanwijzing
Brugsanvisning
Bruksanvisning
Käyttöohje
Návod k použití
Instrukcja obsługi
サービスマニュアル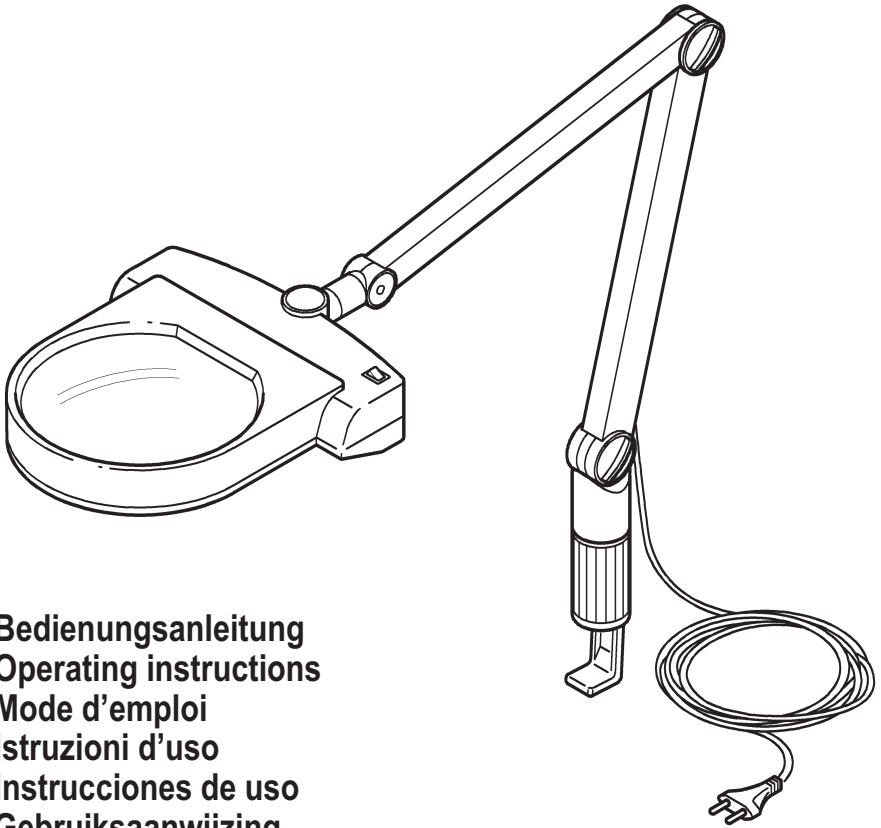

ESCHENBACH

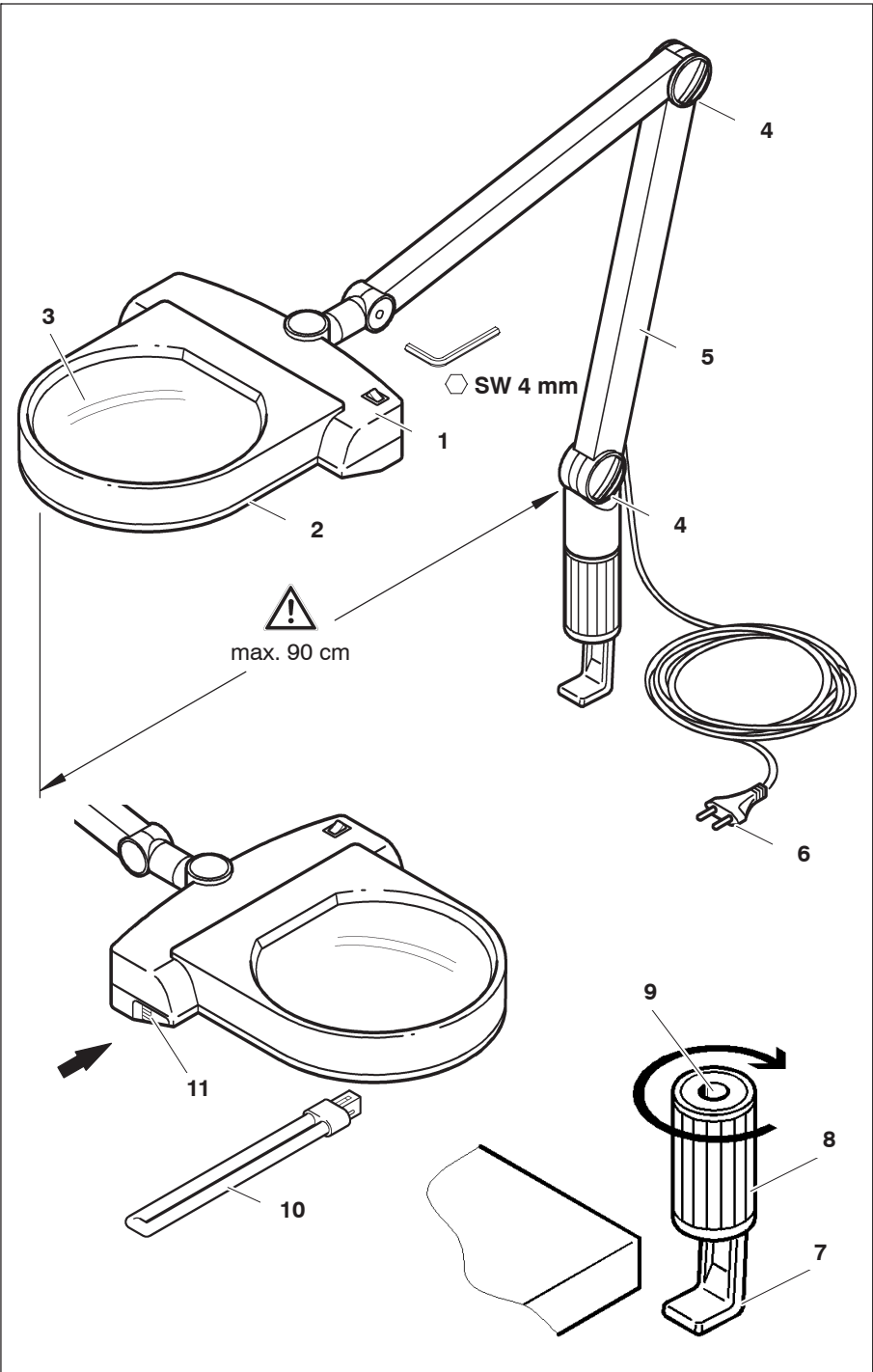Deutsch

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English

Free-standing reading lenses by Eschenbach. For better quality of life.

You have selected a quality product from the house of Eschenbach which has been produced using the latest manufacturing processes in Germany, a top-class „Made in Germany“ product. We congratulate you on your choice.

The lens, made of high-quality, unbreakable PXM® material, is protected with a cera-tec® hard coating. Scratch-resistance is a decisive criterion for the quality and long-life of the lens.

Safety instructions

- **Danger of fire!**
Lenses used in optical equipment may cause considerable damage through their „burning glass effect“ if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!
- **Danger of blinding or injury!**
Never look at the sun through any optical equipment!
- **Protect your free-standing reading lens against jolts and impact shocks, moisture and excessive heat. Never place your free-standing reading lens on heaters or in the sunlight.**
- **The adjusting screws [4] always have to be firmly tightened when using the free-standing reading lens.**
- **Do not use the free-standing reading lens near receptacles containing water!**
- **Danger of tripping!**
Ensure that no-one can trip over the connecting cable!
- **Do not wind the connecting cable around the free-standing reading lens!**
- **To prevent endangerment, a damaged outer flexible cable of the free-standing**

reading lens may be replaced only by the manufacturer, his service representative or an equally qualified technician.

- **Pass on the warning of these dangers to other people!**

Assemble free-standing reading lens

- [1] ON / OFF switch
- [2] Housing
- [3] Lens
- [4] Adjusting screw
- [5] Articulated stand
- [6] Mains plug with 3 m mains cable
- [7] Table clamp
- [8] Clamping cylinder
- [9] Bore for articulated stand
- [10] Lamp
- [11] Lamp cover

1. Screw the table clamp [7] down to a table top using the clamping cylinder [8].
 2. Insert the articulated stand [5] into the bore [9] in the table clamp.
 3. Plug the mains plug [6] into a power socket.
 4. Using the switch, [1] switch the lamp on.
 5. Adjust the free-standing reading lens to the appropriate working / reading height.
- It is normal for the lamp to become slightly warm. The plug can be left in the mains socket even after the lamp has been switched off.

Changing the bulb

1. **Pull the plug [6] out of the socket.**
2. To release the lamp cover [11], press in the direction of arrow and remove the lamp cover by pulling downwards.

- Only use bulbs specified in the technical data.
Replacement fluorescent tubes can be obtained from specialist dealers in optical equipment.
- 3. Change the defective bulb for a new one [10].

Instructions for care

● Pull the plug [6] out of the socket.

Clean the housing and stand with a damp cloth.

Clean the lens with a soft, lint-free cloth (e. g. spectacle cleaning cloth).

For harder to remove dirt (e.g. fingerprints), clean the lens carefully with a damp cloth.

Do not use any soap solutions that contain softeners, alcohol-based solvents or abrasive cleaning products. These may damage the lenses.

Technical Data

Free-standing reading lens:	:	Order no. 2771 1
Diameter:		220 mm
Magnification:		3,0 dpt. / 1,75x
Bulb:		Best.-Nr. 1603 2
Mains voltage:		230 V, 50 Hz
Working radius of articulated stand:		900 mm

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Garantee

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH