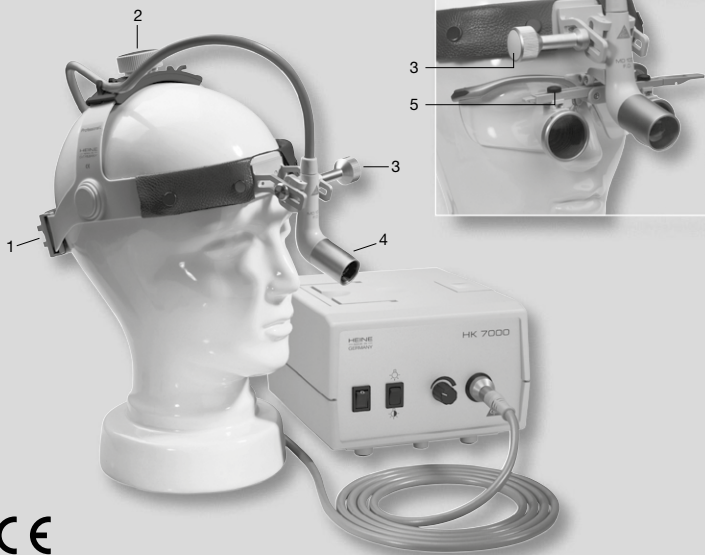
HEINE QUALITY
MADE IN GERMANY

HEINE® MD 1000 F.O. Headlight

CE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 3801 2013-12-16

V-200.00.027

HEINE® MD 1000 F.O. Headlight

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	20
SUOMI	23
PORTUGUÊS	25

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Please read and follow these instructions for use and keep them for future reference.

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Intended Use

The HEINE® MD 1000 headlight is used for examination and during treatment.

OPERATE

Adjusting the beam direction

- Plug in the fiber optics cable to your light source and switch on.
- Place the instrument on your head and adjust for width (1) and height (2) to that it fits comfortably.
- Loosen the fixing screw (3) to adjust the light head up and down and backwards or forwards.
- After adjustment, re-tighten the screw (3).

Selecting viewing angle and field of view

Without HEINE S-Guard®

Loosen the fixing screw (3) to adjust the light head up and down and backwards or forwards.
After adjustment, re-tighten the screw (3).

With HEINE S-Guard®

- Before putting on the loupes, select your inter-pupillary distance (PD) or an average value on the scale. Unscrew the retaining screw (5) and slide the eyepieces to one side symmetrically.
- Unscrew the fixing screw (3) and locate the nosepad on the bridge of the nose.
- Adjust the eyepieces so that they nearly touch the protective lenses.
- Shine the MD 1000 onto an object at your normal working distance and adjust the eyepieces so that the object is centred in both eyepieces. After adjustment, re-tighten the screw (3).

The loupes are correctly adjusted when you see a clearly-defined, round, shadow-free field of view. Please consult the Instruction leaflet for HEINE® HR and HRP binocular loupes.

The illuminated field of view can be selected by means of the adjusting ring (4).

The adjusting ring can be fitted with a sterilisable sleeve.

Fitting HEINE binocular loupes and a video camera: Use the appropriate adaptor, which is available as an accessory.

Supporting the fiber optics cable

The F.O. cable includes adjustable clips which are used to attach the cable to a gown or similar.

HYGIENIC CLEANING / PROCESSING

Use a micro-fiber cloth to clean the lens. The instrument housing can be cleaned with a soft cloth moistened with alcohol or a non-aggressive disinfectant solution. Do not soak or spray with disinfectant.

The adjuster sleeve can be autoclaved at 134°C for 5 minutes.

Read the instructions for cleaning the HEINE® i-View

Maintenance

This product does not require regular maintenance.

Service

This product does not require regular service.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Technical specification

Weight:	ca. 230 g (cable supported by clips)
F.O. cable length:	300 cm
Diameter of light spot at 50 cm distance:	approx. 40 – 160 mm
Type of operation:	continuous
Ambient temperature:	+10°C to +40°C
Classification:	moderate risk, IEC 62471 [2006]

General Notes and Warnings

-  Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.
 - Use only light sources which are approved for use in a medical environment.
 - Do not shine the light directly into the patient's eyes.
-  This product is a precise optical instrument. Please handle this product with care and store it in a clean place.
-  This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

	The CE mark indicates that the product complies with the European medical device directive 93/42/EEC.
	Catalogue- or reference number
	Serial number
	Manufacturer
	Date of manufacture
	Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE)
	Temperature limits in °C for storage and transport
	Temperature limits in °F for storage and transport
	Humidity limitation for storage and transport
	Please read and follow the instructions for use and keep them for future reference (background color blue; foreground color white)
	Fragile, handle with care!
	Keep dry
	Interference may occur in the vicinity of equipment marked with the following symbol
	"Grüner Punkt" (country-specific)

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