
HEINE QUALITY
MADE IN GERMANY

HEINE Headlights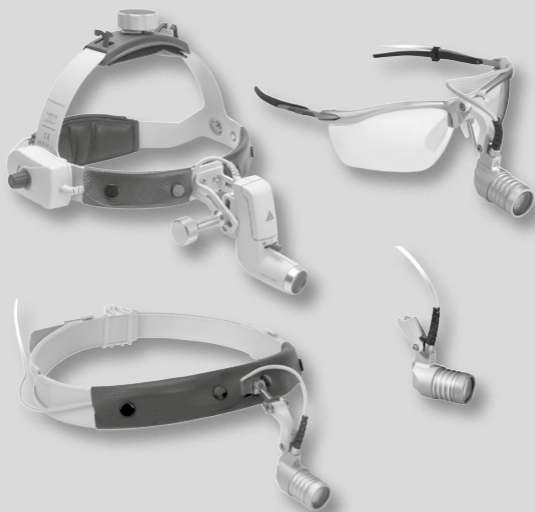

CE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 0714 2014-12-02

V-200.00.033

HEINE Headlights

DEUTSCH	3
ENGLISH	9
FRANÇAIS	15
ESPAÑOL	21
ITALIANO	27
SVENSKA	33
NEDERLANDS	39
DANSK	45
SUOMI	51
PORTUGUÊS	57

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HEINE Headlights

These instructions apply to the following products of the HEINE Headlight series: ML4 LED, LED LoupeLight, LED MicroLight.

Please read and follow these instructions for use of and keep them for future reference. (Background: blue, foreground: white.)

Intended Use

HEINE headlights are battery- or mains-operated devices, which are designed to provide light during diagnosis or during a therapeutic or surgical (only ML4 LED) intervention. They must be used only by authorised qualified medical personnel. Do not shine the light into eyes.

For U.S. only:

 Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Starting-up

Only operate the **ML4 LED** with the HEINE® mPack, the mPack UNPLUGGED or the plug transformer for the headband rheostat.

Only operate the **LED LoupeLight** and the **LED MicroLight** with the HEINE® mPack LL.

Take care that the cable is fastened tightly to the headband of the ML4 LED in the provided buckles (see Fig. 1).

Fig. 1 Safe cable routing headband ML4 LED

Ensure safe cable routing (see Figs. 2 and 3) for LED LoupeLight and LED MicroLight. Avoid high mechanical loads and kinking of the electrical supply cable of the headlights. Only use the HEINE plug transformer (Cat.No. X-095.16.330) for the HEINE® ML4 LED.

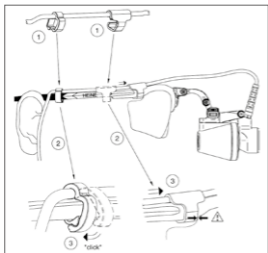
Fig. 2 Safe cable routing S-frame

Fig. 3 Safe cable routing headband Lightweight

Adjustment of the **headband Professional L** (see Fig. 4): To adjust for the head size, the headband can be widened or tightened by turning the turnbuckles at the top (1) and the back (2).

Adjustment of the **headband Lightweight** (see Fig. 5): To adjust the head size, the headband can be widened by pressing the lever (1) and tightened by pulling at the free end (2).

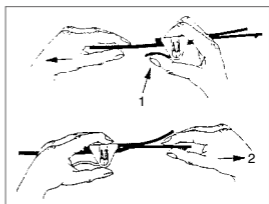
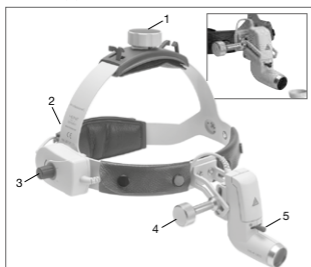
Fig. 4 Headband Professional L (see tall fig.), optical protection cover (see small fig.)

Fig. 5 Headband Lightweight

ML4 LED, fitting the headband rheostat

You can fit the headband rheostat (1) either on the left or the right side of the headband (see Fig. 6). The headband rheostat (1) can then be used for stepless adjustment of the brightness.

When using the ML4 LED, it is recommended that the mPack rheostat is set to „0“.

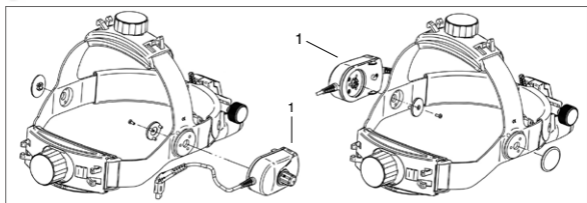
Fig. 6 Fitting the headband rheostat

How to fit the LED LoupeLight to the binocular loupe mount with HEINE i-View® (Fig. 7):

Unscrew the screw (1) on the light mount (2) so that this can be attached to the round attachment of the loupe mount (3). The LED LoupeLight housing has to fit between the eyepieces (not above them!).

The cable for the **LED LoupeLight** can be fixed with the cable clip on the right or left side of the frame as desired.

Fig. 7 Fitting the LED LoupeLight with HEINE i-View®

Operation

HEINE® ML4 LED

To set up the HEINE® ML4 LED with HEINE® mPack or HEINE® EN50, connect the short cord from the brightness control with the connecting cord mPack UNPLUGGED/ Cinch and fix this to the back part of the headband (Fig. 8). Connect the instrument to the Cinch-socket of the mPack or EN 50. The rheostat (1) (Fig. 6) is used to adjust the brightness and to turn the device on and off.

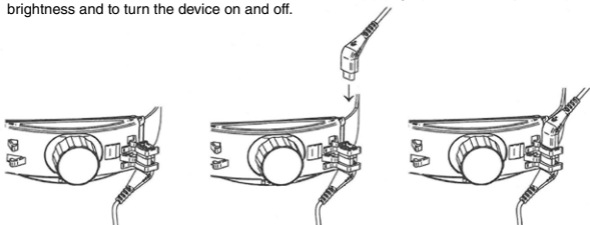
Fig. 8

To use the ML4 LED with the HEINE® mPack UNPLUGGED first remove the rear cover of the headband (Fig. 9).

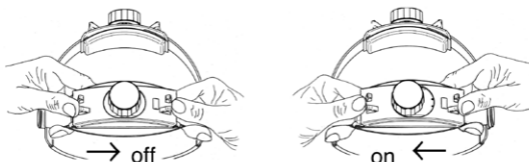
Fig. 9

Then attach the mPack UNPLUGGED to the rear of the headband. To do so, hold the headband in your left hand and push the mPack UNPLUGGED to the left, until it clicks into place (Fig.10).

Fig. 10

Selecting viewing angle and field of view (Fig. 4)

Without HEINE® S-Guard

- After loosening the locking screw (4), the light head can be inclined and the angle of the light direction can be adjusted.
- After adjustment, re-tighten the screw (4).

With HEINE® S-Guard

- Before putting on the loupes, select your inter-pupillary distance (PD) or an average value on the scale. Unscrew the retaining screw and slide the eyepieces to one side symmetrically.
- Unscrew the fixing screw (4) and locate the nosepad on the bridge of the nose.
- Adjust the eyepieces so that they nearly touch the protective lenses.
- Shine the ML4 LED onto an object at your normal working distance and adjust the eyepieces so that the object is centred in both eyepieces. After adjustment, re-tighten the screw (4).
The loupes are correctly adjusted when you see a clearly-defined, round, shadow-free field of view. The slider (5) allows up or down adjustment of size of the light field.

HEINE® LED LoupeLight and HEINE® LED MicroLight

- Connect the LED Light to HEINE® mPack LL.
- Switch on the LED Light and adjust vertically the light housing such that the illumination is centred on the area to be examined.
- Turning the swivel lever at the mPack LL allows continuous adjustment of the brightness.
- Fasten the cable on your clothes using the support clip provided (see Fig. 11).


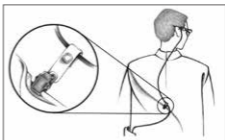
Fig. 11 Cable fastening with support clip

LED LoupeLight

- Adjust the binocular loupe for the intended application. Be sure the eyepieces are positioned as symmetrically as possible on the optics holder (take off the loupe and check this), as otherwise the spot of light will appear shifted to the side.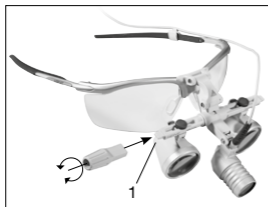

- **LED LoupeLight on loupe mount with HEINE i-View®.** If required, a steri-lever (12) can be attached to one end (1) of the loupe mount so that the binocular loupe can be swivelled independently of the light (only HR/HRP Loupes).

Fig. 12 Using the steri-lever

LED MicroLight

- In order that the LED MicroLight remains in place in every gradually and freely selectable position, it may be necessary to retighten the screws using a screwdriver (see Fig. 13).

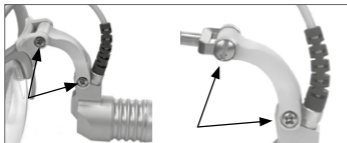
Fig. 13 Tighten the screws

Consumable material and options

For applications in the dental area:

If necessary, use a yellow filter in order to avoid the composite filling material hardening, as this blocks UV radiation.

The plug-on polarisation filter P2 enables a differentiation and low-reflex examination of tissue surfaces due to polarized light. Only in combination with ML4 LED light head or LED LoupeLight and HR 2.5x binocular loupes, ready to use. The polarisation filter P2 is not a replacement for a yellow filter as it may harden composite filling material.

The setup and operation of the HEINE power sources are described in a separate instruction document.

Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: noncritical

Spaulding Classification USA: noncritical

 Allow the device to cool down before reprocessing.

Before cleaning, disconnect the device from the power source.

Before you plug the device back in, dry the plugs of the LED LoupeLight and the LED MicroLight.

In the event of suspected contamination, carry out hygienic preparation of the instrument.

The device and its accessories are not sterile. Avoid touching with sterile hands, areas or instruments.

The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the resources and procedures named in these instructions for use.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.

Do not use any spray disinfectant, or any dripping-wet or heavily-foaming cloths.

Do not prepare mechanically.

For the polarisation filter P2, do not use any alcoholic preparing agents.

Procedure

Clean and disinfect the head lights, the options and the headbands manually (wipe-cleaning and wipe-disinfecting). Clean the leather cushion of the headbands with a detergent cloth. Conditioning residues on optics may be removed using a low-bacterial, dry, micro-fibre cloth.

Recommended agents:

Cleaning agent: Neodisher® MediClean

Disinfectant agent: quaternary ammonia compounds (e.g. Microbac® tissues)

Please follow the cleaning notes for the S-Frame in the identically named instructions for use.

Number of preparation cycles

Polarisation filter P2 may be re-prepared up to 50 times.

Maintenance and Service

The instruments do not require maintenance or service.

The expected operational life of the HEINE head light is 5 years.

General Warnings

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments)

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Danger from loose, falling parts. Please check before every use that all the parts are tightened safely.

Route the supply cable in such a way that nobody can trip over it.

Do not shine the light directly into the eyes. Never ever hold magnifying glasses or any other optical system in front of the light of the LED-headlight.

Only use the lens protector as transport protection. Store the lens protector in the transport case. Do not use in the surgical rooms.

Take note of the operating time of the rechargeable batteries. Keep a charged back-up rechargeable battery ready if necessary.

Jerky, fast movements can alter the position of the lamp housing.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com. Store and use the device in dry and dust-free environments only!

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

 Recycle batteries separately.

Electromagnetic Compatibility

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

 This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.

The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices.

If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains the tables „Guidance and Manufacturer's declaration – Electromagnetic immunity“, technical specification and an explanation of the used symbols.

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Guidance and manufacturer's declaration – electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such environment.

Emission test	Compliance	Electromagnetic environment – Guidelines
RF emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely that any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	not applicable	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Passed	

Guidance and manufacturer declaration - Electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – Guideline
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for mains cables ± 1 kV for input and output lines	± 2 kV for mains cables ± 1 kV for input and output lines	The supply voltage quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV voltage phase – phase, ± 2 kV voltage phase – earth	± 1 kV voltage phase – phase ± 2 kV voltage phase – earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT, (>95% dip in UT) for 1/2 period 40% UT, (60% dip in UT) for 5 periods 70% UT, (30% dip in UT) for 25 periods <5% UT, (>95% dip in UT) for 5 seconds	<5% UT, (>95% dip in UT) for 1/2 period 40% UT, (60% dip in UT) for 5 periods 70% UT, (30% dip in UT) for 25 periods <5% UT, (>95% dip in UT) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered by a UPS (uninterruptible power supply) or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Comment: UT is the a.c. supply voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment Guidelines
Conducted RF IEC 61000-4-6	3 Veff 150 kHz to 80 MHz	3 V eff	Portable and mobile RF communication equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated HF IEC 61000-4-3	3 V/m 80MHz to 2,5GHz	3 V/m	<p>Recommended separation distance: $d = 3,5/3 * \text{SQRT}(P/W)$ $d = 3,5/3 * \text{SQRT}(P/W)$ 80 MHz to 800 MHz $d = 7/3 * \text{SQRT}(P/W)$ 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note 1: At 80Hz and 800MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy.

To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V / m.

Recommended separation distances for portable and mobile RF communication equipment and the device

The device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 3,5/3 * \text{SQRT}(P)$	80 MHz to 800 MHz $d = 3,5/3 * \text{SQRT}(P)$	800 MHz to 2,5 GHz $d = 7/3 * \text{SQRT}(P)$
0.01	0.1	0.1	0.2
0.1	0.4	0.4	0.7
1	1.2	1.2	2.3
10	3.7	3.7	7.4
100	11.7	11.7	23.3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Technical specification

Nominal voltage ML4 LED LED LoupeLight/LED MicroLight	6-9V DC 3V DC
Nominal current ML4 LED LED LoupeLight/LED MicroLight	max. 600mA 350mA
Protection class	II (power supply) Intern power supply (accu)
Illuminance (without filters) ML4 LED LED LoupeLight/LED MicroLight	≥ 50.000lx ≥ 40.000lx
Colour temperature	typ. 4500K
Colour rendering index (CRI)	≥ 90
(Tied) cable length ML4 LED LED LoupeLight/LED MicroLight	1.60m 1.45m
Light field diameter at a 420mm working distance ML4 LED LED LoupeLight/LED MicroLight	30-80mm ca. 80mm
Vertical direction of illumination ML4 LED LED LoupeLight LED MicroLight	± 20° adjustable ± 7° adjustable ± 90° adjustable
Weight ML4 LED LED LoupeLight/LED MicroLight	70g (light head with rheostat) 50g
Operation time ML4 LED LED LoupeLight/LED MicroLight	8,5h with HEINE® mPack; 3,5h with HEINE® mPack UNPLUGGED 17h with HEINE® mPack LL
Environmental conditions for operation	+10°C to +35°C 30% to 75% rel. humidity 700hPa to 1060hPa
Environmental conditions for storage	+5°C to +45°C 45% to 80% rel. humidity 500hPa to 1060hPa
Environmental conditions for transport	-20°C to +50°C 45% to 80% rel. humidity 500hPa to 1060hPa
Device classification according to IEC 62471	Group: 2

Consumables

LED LoupeLight / LED MicroLight	Sterilisable Swivel Lever for rigid loupe mount (without i-View)	C-000.32.524
	Sterilisable Swivel Lever for loupes with i-View	C-000.32.551
	Yellow Filter	C-000.32.241
	Polarisation filter P2	C-000.32.535
ML4 LED	Yellow Filter	J-000.31.321
	Polarisation filter P2	J-000.31.326
Loupes		
Spare polarisation filter P2 for HR Loupes (2 pairs)		C-000.32.536

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Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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